


3

PINS AND INSTRUMENTS

The Rush Pin Set*

This set is adequate for practically all fractures. Rare individuals require smaller diameters for the forearm. This is unlikely in the aged, whose medullary cavities are usually large.


The pins are of one design for all long bones. However, for maximum surgical efficiency, it has been found that the temper of the rod must differ in the various diameters.

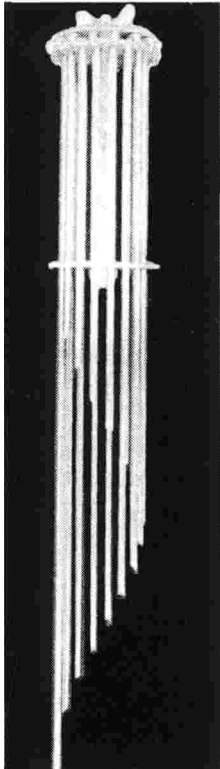

SET 100

Femur, Shaft
Tibia, Shaft
Humerus, Shaft


DIA. 6 mm (1/4")
Length Range –
9 1/2" Thru 20 3/4"

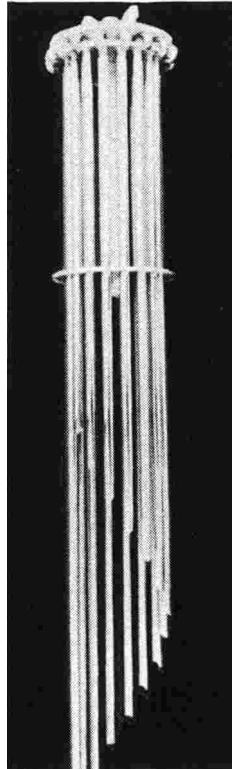

SET 200

Humerus, Shaft –
Small Adults
Femur, Lower Third
Tibia, Shaft,
Condyles

DIA. 5 mm (3/16")
Length Range –
7 1/4" Thru 18 1/2"

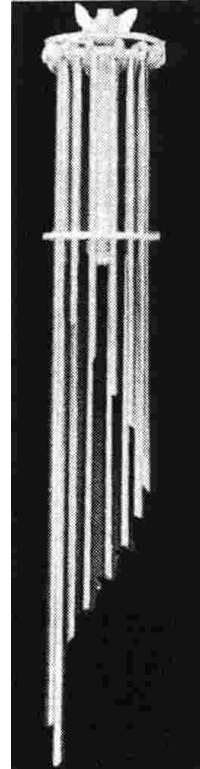


SET 300

Humerus, Condyles
Radius, Shaft Colles
Ulna, Shaft, Olecranon
Clavicle
Tibia, malleolus
Fibula, malleolus
DIA. 3 mm (1/8")
Length Range –
4" Thru 10 1/2"



*STANDARD

The Rush Pin Set* Cont'd.

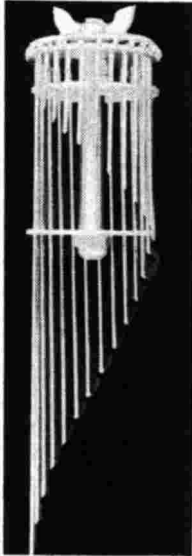

SET 400

Ulna, Radius, *In small individuals*

Malleoli, *Of Tibia and Fibula*

Metacarpals

Humeral, *Condyles*

Acromio-Clavicular Separation

Clavicle

DIA. 2 mm (3/32")

Length Range – 1" Thru 10"



SET 500

Metacarpals

Phalanges

Metatarsals

DIA. 1.6 mm (1/16")


**Length Range –
3/4" Thru 3 3/4"**

RUSH CONDYLE PIN

DETAIL

Condyle Head

Condyle head may be offset for screw or nail usage if desired.

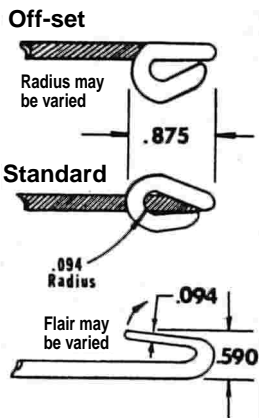

For osteoporotic condyles, the problem has been the migration of the head of the pin into the bone with loss of fixation. This problem has been overcome by the modification of the head to the Rush Pin into a much broader, flatter surface.

The new Rush Condyle Pin ideally expands the Rush Pin Technics to include some previously difficult cases involving the friable or osteoporotic condyles in the elderly.

Particularly useful in supracondylar fractures of the femur and the condyles of the tibia. Same procedures as Rush Pinning.

NO. 250

DIA. - 5 mm (3/16")


LENGTH RANGE - 7 1/4" THRU 18 1/2"

CUSTOM LENGTHS AVAILABLE

“LUXURY” INSTRUMENTS*

The instruments facilitate the operation and make it more intriguing for the surgeon. A good aluminum mallet and a pair of pliers are essential. The pin extractor is a great help in removing pins of the larger sizes.


RUSH INDEX RACK

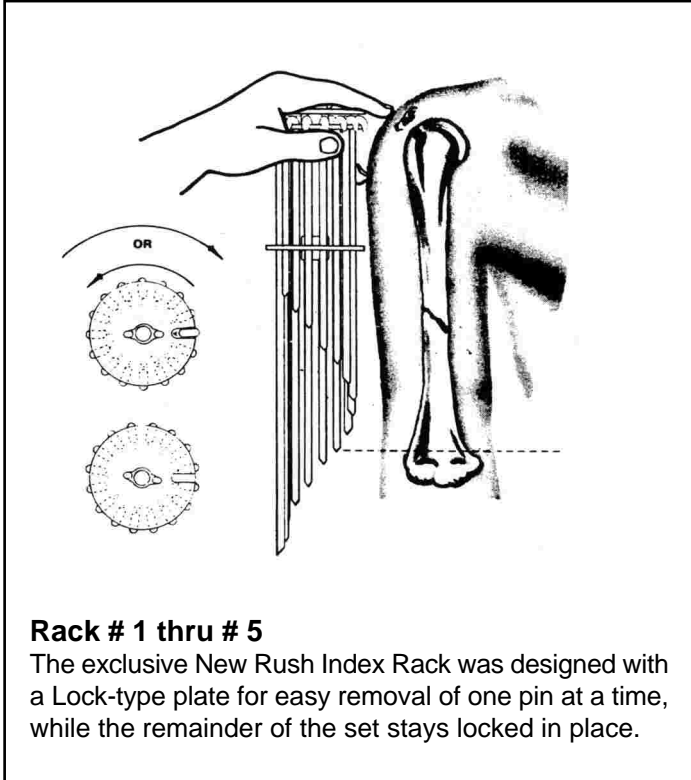

Rack # 1 thru # 5

The exclusive New Rush Index Rack was designed with a Lock-type plate for easy removal of one pin at a time, while the remainder of the set stays locked in place.

RUSH AWL REAMERS

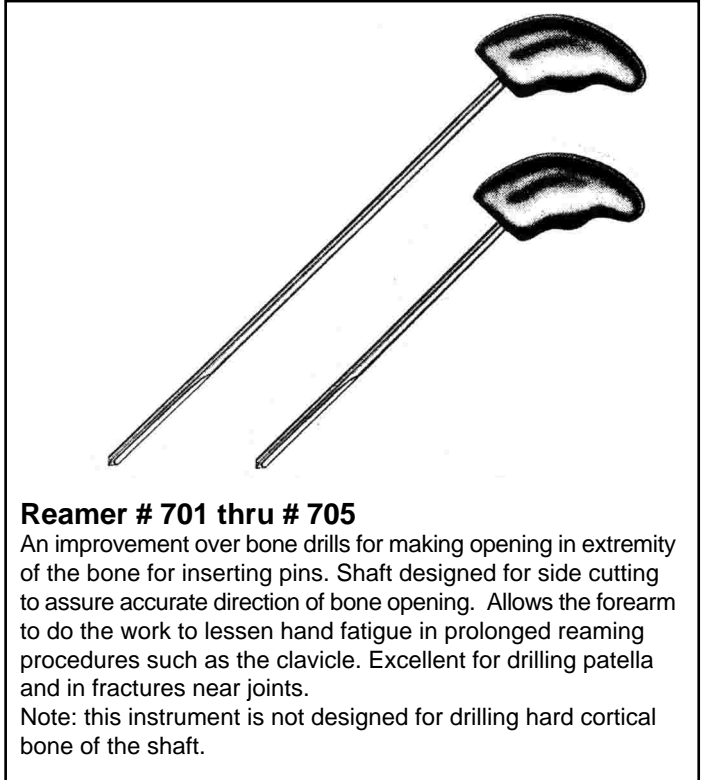

Reamer # 701 thru # 705

An improvement over bone drills for making opening in extremity of the bone for inserting pins. Shaft designed for side cutting to assure accurate direction of bone opening. Allows the forearm to do the work to lessen hand fatigue in prolonged reaming procedures such as the clavicle. Excellent for drilling patella and in fractures near joints.

Note: this instrument is not designed for drilling hard cortical bone of the shaft.


RUSH DRIVER EXTRACTOR



Driver # 751 thru # 755

Cupped extremity fashioned to contour of pin head to prevent bruising of pin. Allows setting the head without trauma to bone or soft tissue. This instrument is not necessary for Rush Pin fixation but greatly adds to the ease of it.

RUSH PIN EXTRACTOR

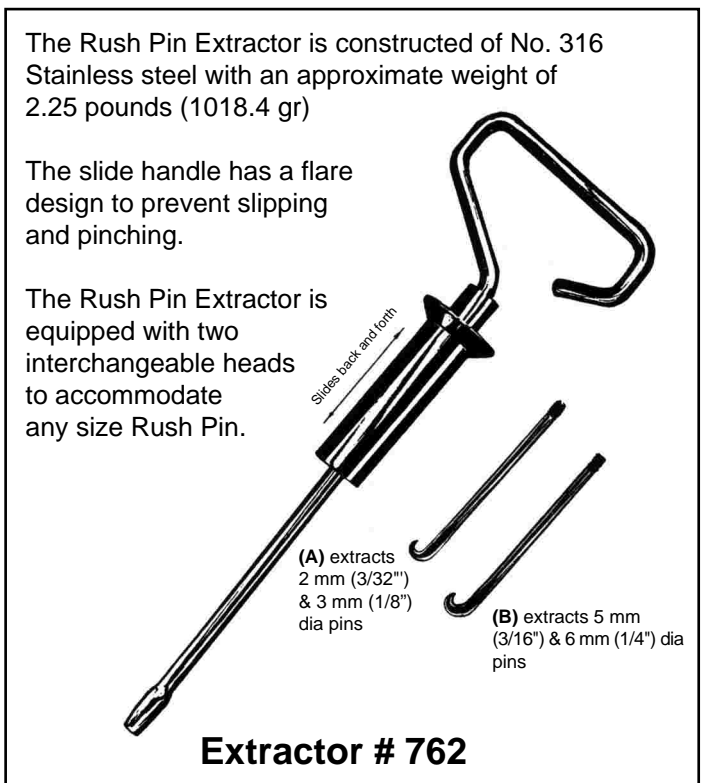

The Rush Pin Extractor is constructed of No. 316 Stainless steel with an approximate weight of 2.25 pounds (1018.4 gr)

The slide handle has a flare design to prevent slipping and pinching.

The Rush Pin Extractor is equipped with two interchangeable heads to accommodate any size Rush Pin.


(A) extracts 2 mm (3/32") & 3 mm (1/8") dia pins

(B) extracts 5 mm (3/16") & 6 mm (1/4") dia pins

Extractor # 762

INSTRUMENTS (cont'd)

RUSH WIRE CARRIERS

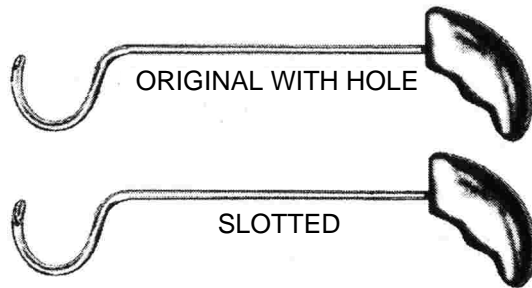


WIRE CARRIER # 801 thru # 8033

Designed to facilitate the placing of circular wires in comminuted fractures with minimal surgical trauma.

Special dissecting tip permits passage through soft tissue attachments while "hugging" bone. Special curve permits passage far enough around bone that wire can be threaded through eye visually. Special non-skid, palm-fit handle for accurate control. Available in slotted or original with hole.

RUSH MALLETS

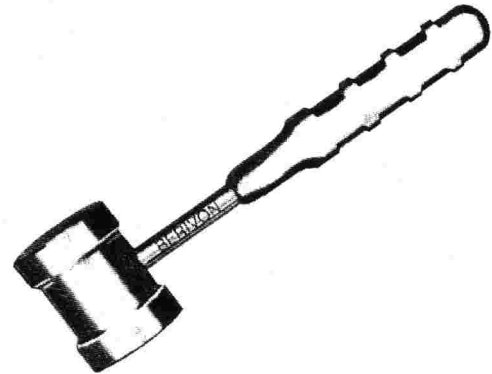

MALLET # 850

Non slip aluminum handle. Aluminum head; 316 stainless steel shaft. Well balanced.

RUSH BENDERS



BENDER # 900

Heavy stainless steel bender for pre-shaping or relieving stress on Rush Pins. 316 stainless steel as a safeguard against corrosion.

RUSH COUNTERSUPPORT



COUNTERSUPPORT # 950

Designed primarily for supracondylar femur to aid reduction and x-raying during the surgery. It has now proved to have many other valuable uses. Pinning of tibia shaft fractures can be greatly facilitated by its use, in both closed or semi-open operations. For the shaft of the femur it is invaluable to permit traction with the flexed knee and to elevate the lower fragment.